

# MEDICAL DEVICES

## Bone Marrow Collection Kit

COLLECTION AND GRAVITY-FLOW  
FILTRATION SYSTEM



**FRESENIUS  
KABI**

caring for life

# Bone Marrow Collection Kit

## Collection and gravity-flow filtration system


### Simple two-step process

The disposable Bone Marrow Collection Kit with flexible prefilter and in-line filters offers simplified collection and filtration of aspirated bone marrow for autologous or allogeneic transplantation.

- Aspirated bone marrow is transferred from syringe to sterile 1.2 liter collection container
- Bone marrow flows through filters by gravity into transfer pack units provided with the kit

The gravity-flow filtration system saves time in the operating room. The system minimizes the need for manipulation during the filtering process. The disposable collection container, filters and transfer pack units are sterile and wrapped for presentation into the sterile field.


The Bone Marrow Collection Kit with flexible prefilter and in-line filters consists of separate components that are assembled with large-diameter connectors for filtration.

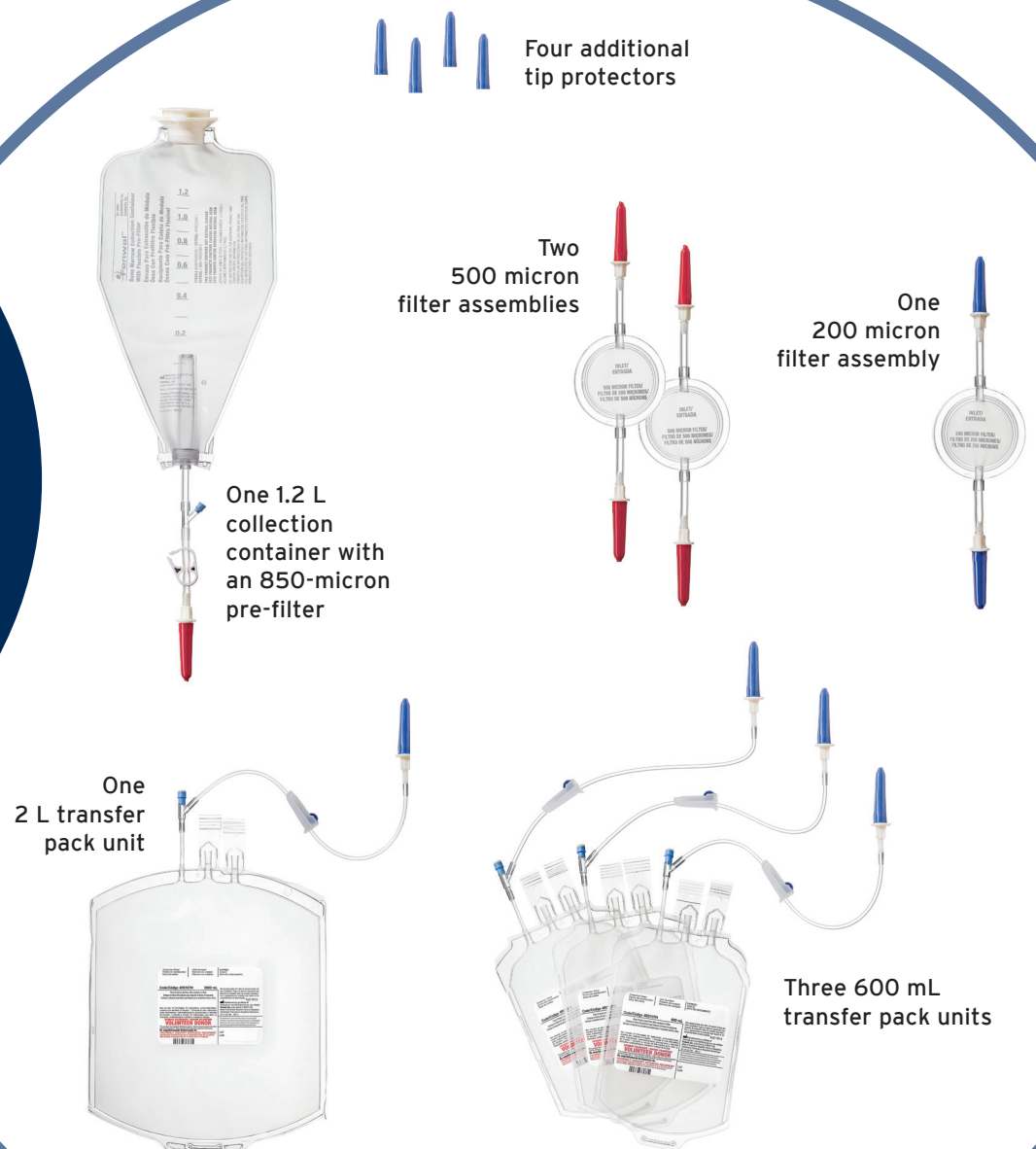

## The Bone Marrow Collection Kit

- **Minimal setup** - includes components needed for collection and filtration in one package (patient-related supplies including aspiration needles, syringes and anticoagulant are not included)
- **Decreases the risk of contamination and spills** - fewer openings than current procedures
- **Easily disposable** - single use plastic components
- **Simple collection** - wide opening on collection container with cap closure
- **Allows for pre- and post- filtration samples** - injection sites on collection containers and transfer pack unit
- **Prefilter removes large aggregates** - 850 micron prefilter reduces likelihood of clogging
- **Gravity flow** - through medical grade with mesh filters; includes two 500 micron filters in case clogging occurs

The reusable Bone Marrow Collection Kit stand is available separately. It has a heavy stainless steel base for stability and is sterilized prior to each use.

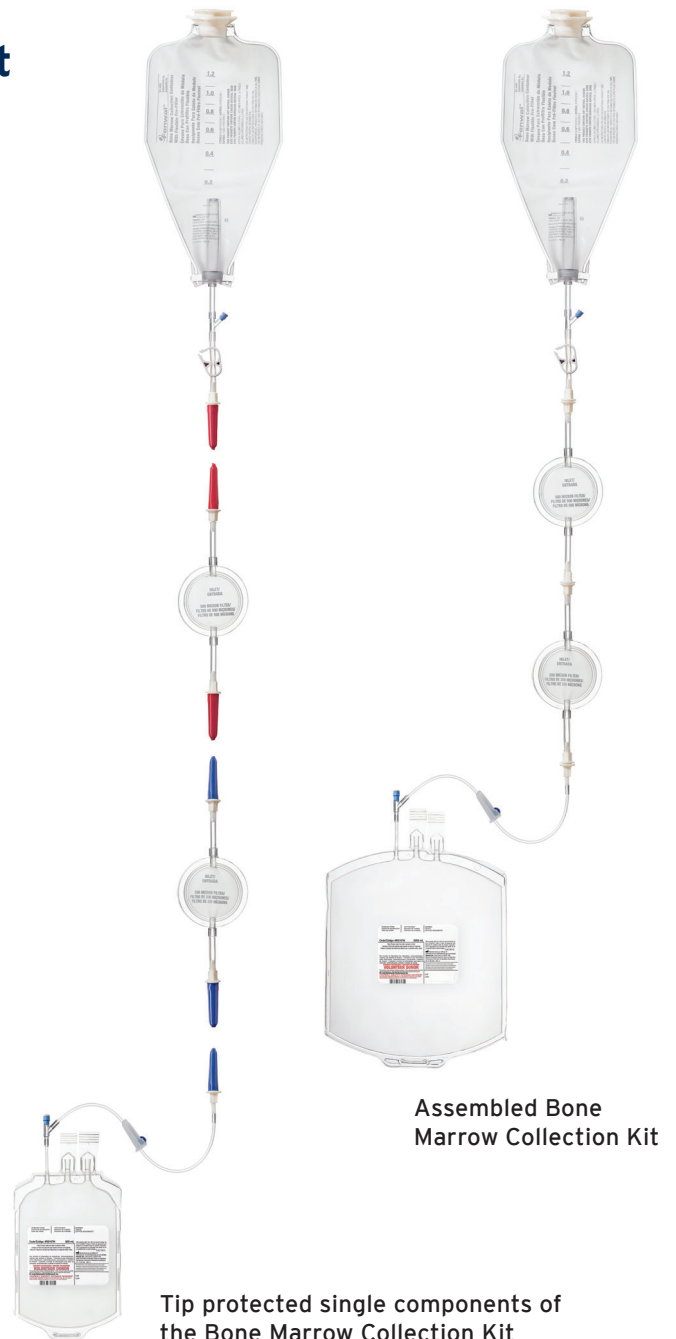

Assembled Bone Marrow Collection Kit

Tip protected single components of the Bone Marrow Collection Kit

## Ordering information

For more information such as literature, technical details and working procedures as well as for equipment please contact your local sales representative or us.

### ORDER NUMBERS

### ORDERING PACKAGE

R4R2107H	Bone Marrow Collection Kit with Flexible Pre-Filter and Inline Filters (CE marked)	1 unit
4R2107J	Bone Marrow Collection Kit with Flexible Pre-Filter and Inline Filters (only for Japan)	1 unit
4R2107H	Bone Marrow Collection Kit with Flexible Pre-Filter and Inline Filters	1 unit
4R2105	Bone Marrow Collection Kit Stand	1 unit



 **FRESENIUS  
KABI**  
caring for life

#### Fresenius Kabi

Else-Kröner-Straße 1  
61352 Bad Homburg, Germany  
Phone: +49 (0) 61 72 686 4912  
Fax: +49 (0) 61 72 686 4728  
bonemarrow@fresenius-kabi.com  
www.fresenius-kabi.com

Your local sales organization: