




**FRESENIUS
KABI**

caring for life

How to Gravity feed using a GraviSet EasyBag giving set

The GraviSet EasyBag giving set (Product code 7751955) is compatible with all Fresenius Kabi EasyBags. Also compatible with the Fresubin® Easy Bottle Adaptor (Product code 7989923) and the HydroBag (Product code 7751102).

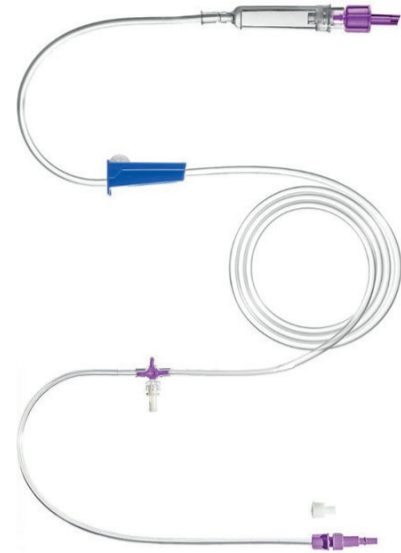

Instructions for use



Close the roller clamp on the giving set.



Spike the EasyBag with giving set.




Hang the EasyBag on the drip stand.



Squeeze the drip chamber until it is half full.



Undo the roller clamp to prime the giving set.

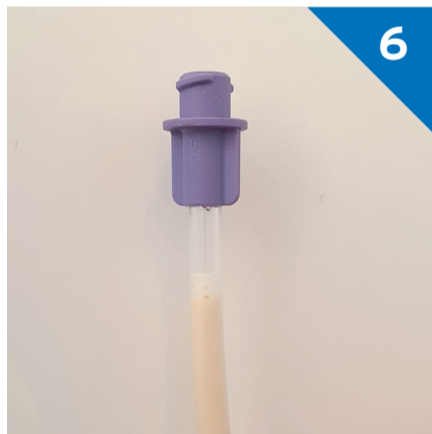

Close the roller clamp when the feed is 1 inch from the end.



Flush the patient's feeding tube with water as per protocol.



Attach the giving set to feeding tube. Control rate of feeding by slowly opening or closing the roller clamp.

Drip rate may vary depending on the viscosity of the feed used. The information below is based on gravity feeding using water as a guide. The standard drop rate is 20 drops per ml.

ml/hr	Drops/minute	Drops/15 seconds
25	8	2
50	17	4
75	25	6
100	34	9
125	42	11
150	51	13

The number of drips has been rounded up to the closest whole number for practical purposes.

Regularly check the drip chamber and feed container to ensure the flow rate is correct and the feeding container does not empty completely. **Discard the giving set after 24 hours.**