



**FRESENIUS
KABI**


caring for life

How to change the Freka® PEG FR15 ENFit® Intestinal feeding tube Y-Connector


Connection of the Y-Connector to the gastric tube

Getting ready

- Always refer to local guidelines
- Collect all equipment needed on a clean table or tray
- Wash and dry hands before and after changing the Freka PEG FR15 ENFit Intestinal feeding tube Y-Connector

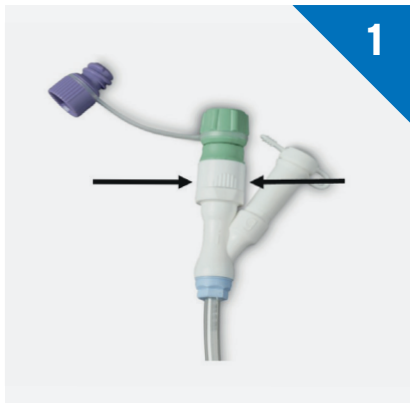


To remove the Freka Click Adapter, squeeze the contoured surfaces of the Click Adapter firmly between two fingers.

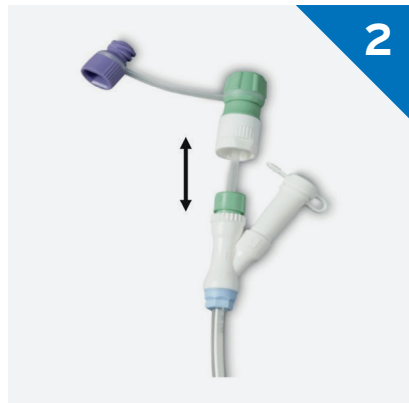

Pull out the intestinal tube carefully for approximately 3 - 4cm.



To prevent the intestinal tube from slipping, close the tube clamp for fixation.

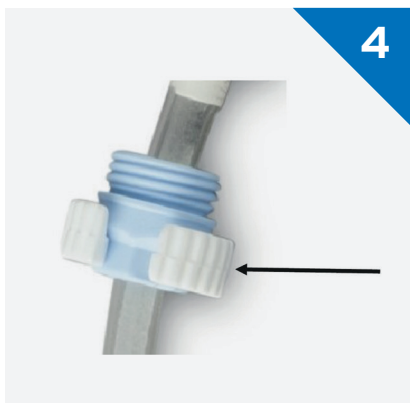


Attach the white screwing aid from the new Freka connector end on to the existing pale blue fixation screw. Unscrew the Freka Y-Connector off the gastric feeding tube and remove the white screwing aid.



With the clamp remaining closed, cut the tube as shown in the image which will remove the Freka Click adapter. Now remove the unscrewed Freka Y-Connector and the pale blue fixation screw.



Attach the white screwing aid on to the new pale blue connector. Push the blue and white fixation screw for the Freka Y-Connector ENFit over the gastric tube as shown.

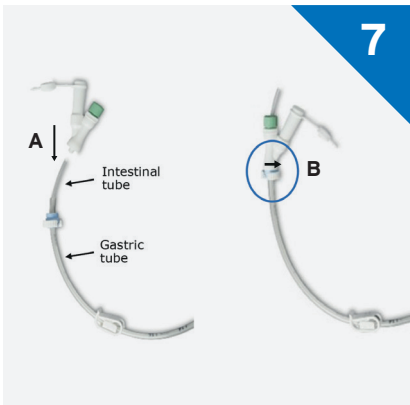

Push the new Freka Y-Connector **as far as possible** over the intestinal tube and into the gastric tube (A). Secure the Freka Y-Connector and the tube with the pale blue fixation screw and the white screwing aid (B). Screw the blue connector and **NOT** the tube.



Remove the white screwing aid (outer white ring) of the Freka Y-Connector.




Push the new Click Adapter (green / white) with its white end first over the intestinal tube.



Push the metal pin of the Freka Connector, up to its base into the intestinal tube.



Screw the Click Adapter (green / white) clockwise and tight onto the Freka Connector. Screw the Click Adapter until no rotation is possible, being careful not to rotate the tube itself.

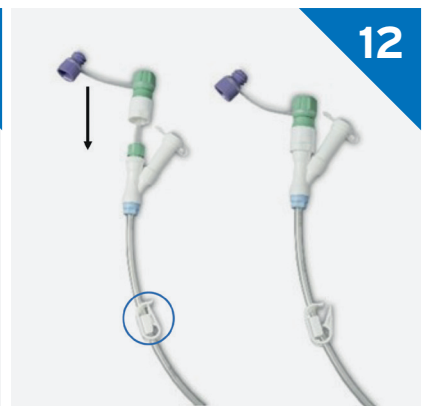

Open the tube clamp of the gastric tube and push the connected Click Adapter over the (green) intestinal access of the Freka Y-Connector, marked with an 'i'. Make sure finger grips are 90 degrees away from the Y port when connected to aid future changing of the Freka Y Connector. The fixation must be clearly audible with a 'click'.



Close the intestinal tube with the cap of the Freka Connector.

Freka PEG Intestinal Feeding Tube Product information	
FREKA CLICK ADAPTOR FOR FREKA PEG INTESTINAL	FREKA Y CONNECTOR FOR FREKA PEG INTESTINAL
<p>DESCRIPTION</p> <ul style="list-style-type: none"> > For Freka PEG Intestinal CH/FR 9 	<p>DESCRIPTION</p> <ul style="list-style-type: none"> > For Freka PEG Intestinal CH/FR 15
<p>Pack of 15</p> <p>Product code: 7981389</p>	<p>Pack of 5</p> <p>Product code: 7981394</p>